

TeltoCare software changelogs

[Main Page](#) > [TeltoCare](#) > **TeltoCare software changelogs**

Changelog

Version	Release date	Changes
Version 1.15.2	Release date 2024-10-09	<p>FEATURES</p> <ul style="list-style-type: none">• Adapted TeltoCare to support various screen sizes, including desktop and mobile.• Improved afib burden bar chart with automatic Y-axis scaling, adding 10% to the max afib burden for better visualization.• Added responsive design to the top navigation bar for better usability across devices.• Implemented responsive design for the data filtering (calendar) component.• Updated the average steps calculation to only include days with data.• Displayed days on the X-axis of the HR data chart when filtering by multiple days, weeks, months, or year.• Enhanced the registration page with a fully responsive design for mobile devices.• Included a link to the login page in account creation emails to streamline user access.• Updated the ECG section for mobile screens by removing the ECG chart for better mobile adaptation.• Maintained consistent ECG visualization across different desktop resolutions.• Set personal info input fields in the profile settings to full width (100%) on mobile devices.• Removed labels from input fields in the profile settings when viewed on mobile for a cleaner design.• Removed labels from input fields in security settings on mobile for a streamlined look.• Adjusted the security settings input fields to full width (100%) on mobile devices.• Enabled automatic file retrieval and uploads to TeltoCare.• Added HR min, max, and average for each entry in the afib section for clearer data insights.• Enabled scheduled file uploads and downloads from the app according to the configured schedule. <p>FIXES</p> <ul style="list-style-type: none">• Fixed ECG chart display issue when zooming in or out.• Corrected non-responsive design on the registration page for mobile devices.• Resolved calendar picker overlapping with navigation menu items on mobile.• Fixed log out button disappearing on small screens.• Corrected incorrect dates on X-axis labels in charts.• Fixed frontend design unresponsiveness on mobile.• Resolved HR data showing misleading months and days when viewed using the year filter.• Fixed missing step count data display.• Addressed duplication issue in the afib section.• Fixed scheduled push triggers for file download/upload not working in production.• Resolved missing heart rate data in the afib section.• Fixed issue preventing new users from logging in due to problems with password changes.• Corrected broken URL in the account invite email.• Fixed issue preventing patient creation using phone numbers.• Resolved incorrect data time display.• Fixed incorrect HR min/max data caused by missing HR entries.
Version 1.13.14	Release date 2024-04-30	

FEATURES

New functionalities

- Added support for user registration via the app
- Simplified registration form to include only mandatory fields for patient creation
- Users with unverified emails are now denied access to TeltoCare
- Users must verify their email after entering patient details to continue registration
- New registration form includes all mandatory inputs for patient creation
- Added a "Register" button on the authentication screen

Optimization

- Improved patient dashboard page load
- Reduced login loading time
- Reduced user page loading time
- Reduced medical data loading time

Improved user delete functionality

- Users with permissions can now delete their own accounts

Main admin changing functionality

- Main admins must transfer their role to another organization's admin before deleting their own account
- Organization's admins cannot delete their own account if they are the only admin in the organization

Change hospital functionality

- Added an unchecked checkbox by default for hospital visibility for patient transfers
- Hidden hospitals will not appear in the "My Hospital" dropdown
- Checkbox added to show a hospital in the patient's transfer dropdown if checked
- Checkbox added for hospital visibility in "Manage Organizations" page
- Added a section in "Manage Organizations" page for hospital contact details
- Accepted transfer patients now appear in the hospital's "Users" page without "Transfer" status
- Hospital doctors/admins can access the transfer request acceptance window via the "Users" page
- Optional "Reason" input field added in the modal for declining hospital transfer requests
- Declined transfer patients no longer appear in the hospital's "Users" page
- Hospital doctors/admins can access the transfer request decline window via the "Users" page
- Patients requesting to join a hospital now show in the "Users" page with "Transfer" status
- Transfer request patients appear in the "Patients" page for hospital admins or doctors
- "In Queue" status added for hospital transfer requests that are pending approval
- "In Queue" or "Declined" status displayed for hospital transfer requests
- Approved transfer requests no longer appear in the hospital transfer request list
- Decline reason field shown in the patient's request status if declined
- Transfer request status shown as "Declined" if denied by the hospital
- Patients can now cancel their hospital transfer requests
- Only one hospital transfer request can be created at a time
- Added "My Hospital" page in the side menu for patients
- Patient accounts can filter HR, ECG, and Afib data by various time frames (H/D/W/M/Y)

Other improvements

- Improved overall system performance and user experience based on feedback

FIXES

- ECG print time and record time are now shown in the same time zone
- Corrected typo in 'Send request'
- Changes in allergies field by Hospital Doctor account are now saved correctly
- Fixed issue where permission changes were not being saved
- Resolved issue with leads being mixed up in exported ECG
- Fixed visibility issue for doctors when accessed by hospital admin
- Completed status field for patients is now properly displayed
- Users can now delete their own accounts
- Fixed issue where ECG export showed notes incorrectly next to aVR
- Export button in ECG section now correctly displays ECG Information
- Improved quality of ECG PDF exports and fixed time axis issues
- Fixed discrepancies in heart rate averages across different interval settings
- ECG record details are now fully visible in exported ECG files
- Users can now create accounts with the appropriate permissions

Version 1.12.23	Release date 2024-03-06	<p>FIXES</p> <ul style="list-style-type: none"> Resolved issue where users could not see updated Terms and Conditions and Privacy Policy information. Fixed login problems preventing users from accessing their accounts. Fixed loading time when selecting a user Improved error messages to make them clearer when logging into an account that is in the process of deletion.
Version 1.12.20	Release date 2024-03-01	<p>FEATURES</p> <ul style="list-style-type: none"> The timezone/color legend will no longer appear on the right side of the HR section if only one timezone has ever been selected for that user. <p>FIXES</p> <ul style="list-style-type: none"> Afib data is now properly displayed in TeltoCare. Doctors can now delete patient accounts successfully. Fixed issue with mixed-up page numbers in ECG exports.
Version 1.12.16	Release date 2024-02-25	<p>FEATURES</p> <ul style="list-style-type: none"> Implement timezone change history functionality Store date and time in UTC+0 timezone Allow users to change their local time zone Functionality for changing actual timezone shall be stored in the Settings page Allow user to select timezone from a dropdown list Display medical data with adjusted timezone offset Display a separate set of HR data columns if the adjustable timezone is changed Differentiate HR data from different timezones using differently colored bars Have a button in Settings page to show timezone changing history Display a table containing historical information of user's timezone changes containing date, time, and the timezone to which the user switched to Display the timezone in ECG selection buttons to indicate in which timezone it belongs Show legend of colors and timezones on the right side of the HR chart section Allow user to change the previously selected timezone for old data <p>FIXES</p> <ul style="list-style-type: none"> Incorrect max value shown in HR section Hovering over HR data columns is not working correctly Yearly filter shows over 365 days in HR section's X axis HR hover not working as intended Misaligned X axis Hospital admin details not shown in Hospital list Events not showing event type anymore TC and PP pages not loading on prod Heart rate interval not working Email check is case sensitive
Version 1.12.10	Release date 2024-01-31	<p>FIXES</p> <ul style="list-style-type: none"> Non human readable error message when logging into a user who's in the process of deletion Unable to create hospital doctor Search showing incorrect results TeltoCare allows saving an invalid year. Error connecting to the database error: Allows inputting something other than numbers (special symbols, letters) in the phone input field. Can create a user by entering symbols in the email field Duplicating events
		<p>FIXES</p> <ul style="list-style-type: none"> TeltoCare user create: Error to select country when country is selected Hospital navigation bar displays double patients option TeltoCare does not allow Doctor to create users. Patient assign does not work