

TeltoCare software changelogs

[Main Page](#) > [TeltoCare](#) > **TeltoCare software changelogs**

Changelog		
Version	Release date	Changes
Version 1.12.20	Release date 2024-03-01	FEATURES <ul style="list-style-type: none">• Implement timezone change history functionality• Store date and time in UTC+0 timezone• Allow users to change their local time zone• Functionality for changing actual timezone shall be stored in the Settings page• Allow user to select timezone from a dropdown list• Display medical data with adjusted timezone offset• Display a separate set of HR data columns if the adjustable timezone is changed• Differentiate HR data from different timezones using differently colored bars• Have a button in Settings page to show timezone changing history• Display a table containing historical information of user's timezone changes containing date, time, and the timezone to which the user switched to• Display the timezone in ECG selection buttons to indicate in which timezone it belongs• Show legend of colors and timezones on the right side of the HR chart section• Allow user to change the previously selected timezone for old data FIXES <ul style="list-style-type: none">• Incorrect max value shown in HR section• Hovering over HR data columns is not working correctly• Yearly filter shows over 365 days in HR section's X axis• HR hover not working as intended• Misaligned X axis• Hospital admin details not shown in Hospital list• Events not showing event type anymore• TC and PP pages not loading on prod• Heart rate interval not working• Email check is case sensitive
Version 1.12.16	Release date 2024-02-25	
Version 1.12.10	Release date 2024-01-31	FIXES <ul style="list-style-type: none">• Non human readable error message when logging into a user who's in the process of deletion• Unable to create hospital doctor• Search showing incorrect results• TeltoCare allows saving an invalid year.• Error connecting to the database error:• Allows inputting something other than numbers (special symbols, letters) in the phone input field.• Can create a user by entering symbols in the email field• Duplicating events
		FIXES <ul style="list-style-type: none">• TeltoCare user create: Error to select country when country is selected• Hospital navigation bar displays double patients option• TeltoCare does not allow Doctor to create users.• Patient assign does not work